

# BE ADAPTIVE EQUIPMENT LLC

P.O. Box 84  
Columbia City, IN 46725  
www.beadaptive.com

Phone 260-244-7031  
Fax: 260-244-4953  
sales@beadaptive.com



## HQ100 - High Quad Set Up

The following is how to assemble our Model HQ100. First start with the seat pan, the vertical bar that we call the T bar and the horizontal bar that we refer to as the L bar. These two should be assembled into the seat pan first, then the main part of the HQ100 can be placed on the 1" vertical post on the rear of the L bar. (On the bottom of the main part of the HQ100 at the rear, there is a hole that will sit right down over that vertical post.) Then the only thing left to assemble is the long arm that has the trigger mechanism on it. You simply stick the white connector on the end of the cord into the right tube of the main part of the HQ100 that has the 1/4" holes in the top of it. Depress the silver button on the long arm so it will slide into the tube, slide it back to one of the 1/4" holes and allow the pin to lock it into place. The white connector will stick out of the back of the right tube and can be plugged into its mating plug just below.

The system runs off the 12 volt DC power. The wire with the fuse should be hooked to the positive terminal on the battery.



**Step 1**



**Step 2**



**Step 3**



**Step 4**